
www.woodleyequipment.com



qLabs[®] Veterinary Coagulation Analyser In-Practice point-of-care coagulation testing



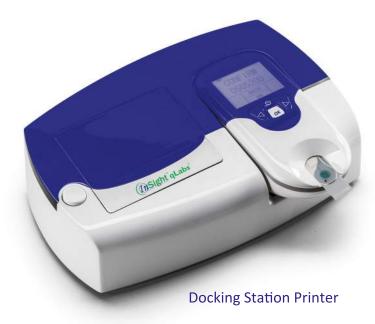

Features:

- Requires 1 drop of non-anticoagulated whole blood (10µl)
- PT/APTT tested simultaneously on 1 test strip (PT only test strip also available)
- Room temperature test strip storage
- · Fill test strip directly from syringe
- QC material available
- No need to transfer sample into citrate tube
- Option for Bluetooth docking station printer
- · Low cost analyser and consumables
- · 'Test count up' for immediate assessment
- · Excellent correlation to central laboratory results

2 Year Warranty

Tel: +44 (0)1204 669033 Fax: +44(0)1204 669034 Email: sales@woodleyequipment.com Web: www.woodleyequipment.com

OLD STATION PARK BUILDINGS, ST. JOHNS STREET, HORWICH, BOLTON, LANCASHIRE, BL6 7NY, UK

Simple Test Procedure:

- 1. Insert test strip
- 2. Apply a drop of blood
- 3. Results displayed on LCD screen

Calibrated and validated specifically for use in cats and dogs



JIPMENT COMPANY

www.woodleyequipment.com



qLabs[®] Veterinary Coagulation Analyser In-Practice point-of-care coagulation testing

- PT/aPTT combo cartridge (12) • PT cartridge (12)

Reference Intervals:

Canine: • PT: 14 to 20 seconds • aPTT: 94 to 123 seconds

Feline:

• PT: 13 to 20 seconds • aPTT: 96 to 122 seconds

Precision:

 In-house lot-to-lot and day-to-day precision <5.0 CV% (mean value)

External QC Control:

- Lyophilised whole blood preparations
- Level I, Level II



Helping diagnose conditions in relation to:

- Pre-operative checks for at risk animals
- Toxins and systemic diseases
- Hereditary bleeding disorders
- Excessive bleeding
- Disseminated Intravascular Coagulopathy (DIC)

Simple Test Procedure:

- 1. Insert test strip
- 2. Apply a drop of blood
- 3. Results displayed on LCD screen

Kit Contents:

- qLabs[®] Coagulation Analyser
- Power Adapter
- Lancets (10)
- Lancing Device
- User Manual
- Carry Case
- Data Manager Installation DVD
- Data Manager Operation Guide
- eCable

2 Year Warranty



Tel: +44 (0)1204 669033 Fax: +44(0)1204 669034 Email: sales@woodleyequipment.com Web: www.woodleyequipment.com

OLD STATION PARK BUILDINGS, ST. JOHNS STREET, HORWICH, BOLTON, LANCASHIRE, BL6 7NY, UK